


Comply™ (SteriGage™) Steam Chemical Integrator

Result Interpretation Chart

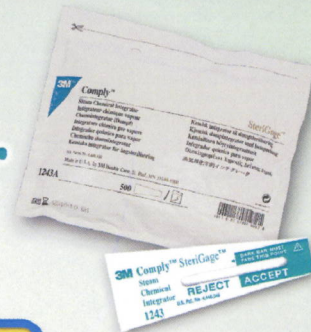

PASS 合格

當色線達到接受水平顯示乎合滅菌所需條件

The color bar has reached the **ACCEPT** window in all three samples shown above, indicating that the necessary conditions for sterilization have been met.

FAIL 不合格

當色線停留於不接受水平顯示不乎合滅菌所需條件

The color bar is in the **REJECT** window, indicating that the necessary conditions for sterilization have not been met.

This testing resulted in the following mean times to produce an ACCEPT reading for each of the seven temperatures tested:

Temperature °F (°C)	Time (minutes)
245°F (118°C)	21.0
250°F (121°C)	14.0
255°F (124°C)	10.0
260°F (127°C)	6.0
265°F (129°C)	3.4
270°F (132°C)	2.0
275°F (135°C)	1.4

