Specifications

	-	
	■ Input	
ı	Input Impedance	10 MΩ or more
	Leads	Standard 12 leads and Cabrera leads (Leads acquisition: Coordinate phase or Real time)
	Patient Input	One patient
	Common Mode Rejection	100 dB or more
	Polarization Voltage	± 500 mV or more
	Frequency Response	0.05 to 150 Hz, within -3 dB to +0.5 dB
	Time Constant	3.2 sec. or more
	Filter	Digital filter, hum noise · drift (weak/normal), Muscle (35/25 Hz)
	A/D Conversion	12 bit (4.88 μV/digit)
	Sampling Rate	1,000 sampling/sec. Pacemaker detection auxiliary function 10,000 samples/sec.
ı	■ Display	
	Display System	5.7-inch STN liquid crystal monochrome display with backlight, 320 × 240 dots
	ECG Display	3Lead, 3+Rhythm Lead, 12Lead
■ Built-in thermal printer		
ı	Recording System	Thermal Array (8 dot/mm)
	ECG Recording Format	1Lead, 1Lead+Rhythm, 3Lead Recording time: 8 to 24 second, Heart rate, Measurement value
	Recording Sensitivity	2.5, 5, 10, 20 mm/mV, Auto selection
ĺ	Recording Speed	5, 10, 12.5, 25, 50 mm/sec.
	ECG Paper	Roll paper (63 mm in width)

Standard Accessories

Power cable	KP300 KS16A for 100V	1
	or VM0307B-VM0303B 2.1M NON PB for 200V	
Limb electrode	CR-3000 (4 pieces/box)	1
Chest electrode	CE-10 (6 pieces/box)	1
Patient cable	PC-201	1
ECG electrode cream 70g		
Operation manual in CD-R		1
Recording paper	R63x30R-C2	1
Paper spool		1

■ Safety

Safety Class I, Type CF Water Proof Protection Non water proof device (IPXO)

Flammable Anaesthetic Not applicable Mode of Operation Continuous operation

■ Power Source

Rated Power Supply 110 to 240 VAC, 50 to 60 Hz, 90 VA

Rechargeable Battery 12 VDC 1500mAh, Nickel-Metal Hydride battery 3 hours or 700 ECG recordings on a full charge

of battery for 8.5 hours. Auto mode: 10 seconds normal waveform and

HR 60.

■ Size Dimensions 266 mm (W) × 173 mm (D) × 75 mm (H) Weight Approx. 1.5 kg (including the rechargeable battery)

■ Operating Environment Temperature

25 to 95% RH (Without dew condensation) Humidity Pressure 700 to 1060 hPa ■ Transportation and Storage Environment

Temperature -10 to 60°C Humidity 25 to 95% RH (Without dew condensation)

Pressure 700 to 1060 hPa ■ Communication Port

IEEE 802.3u 100BASE-TX × 1 LAN port USB port 1 port (USB 2.0 compliant)





Cardico 306 3 Channel Digital Electrocardiograph with Analysis



Cardico 1211 12 Channel Digital Electrocardiograph



Cardico 601 6 Channel Digital Electrocardiograph with Analysis



12 Channel Digital Electrocardiograph with Analysis



Cardico 1215 12 Channel Digital Electrocardiograph



Cardico PC 1201 PC-Based Electrocardiograph



SUZUKEN COMPANY LIMITED

8 Higashikataha-machi Higashi-ku, Nagoya Aichi 461-0015 JAPAN

TEL: +81-52-950-6327 FAX: +81-52-962-7440 http://www.suzuken-kenz.com





^{*}These specifications are subject to change for quality improvement without notice.





Auto Mode

Simple one touch operation of fully automatic ECG recording in preset format.

Pre-Check Mode


Standing by for occurrence of arrhythmia. Provided arrhythmia is detected, ECG recording will be automatically started.


Manual Mode


Conventional manual ECG recording. Leads, speed, sensitivity and filter can be selected manually.

Long Term Mode

Recording a selected lead for a long term (preset for 1 to 5 minutes or continuous recording time).

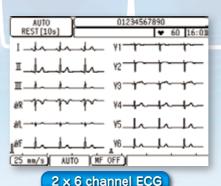

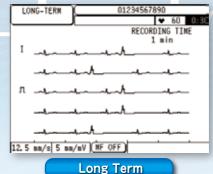

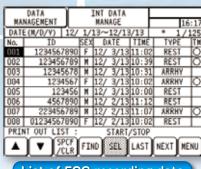

List of ECG recording data

Features



Compact and lightweight

Weighing only 1.5kg, inclusive of a built-in rechargeable battery for superior portability.



5.7 inch touch screen to display a full alphanumeric keyboard to enter patient information and preview of 12 lead ECG waveforms. And an optional external PC keyboard can be connected to the ECG 305.





Large data storage capacity

Internal memory for maximum 200 resting ECG and optional removable USB memory media.



Data transmission

ECG data with patient information can be transmitted via LAN port. Export in XML data format is possible.



Diverse recording formats

Single channel, Single channel + rhythm lead, 3 channels and long-term recording. Maximum 24 seconds of ECG per lead can be recorded.



ECG measurements can be output to support physician's diagnosis.



Barcode and magnetic card scanning options

Optional barcode or magnetic card reader can be connected.

Printed reports

